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HBV RQ Kit Manual

**For Real-Time PCR Detection and
Quantitation of Hepatitis B Virus
(HBV) DNA**

For use with Rotor-Gene or StepOne
Research use only

NG-WI-ASL-01-401
Version 4.1
3 June 2018

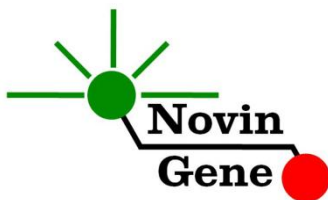
Table of Contents:

1. Introduction.....	2
2. Kit Contents	3
3. Storage and Stability.....	3
4. Additionally Required Materials.....	3
5. General Precautions	4
6. Specimen, storage and transport	4
7. Interfering substances	4
8. DNA isolation	5
9. Internal control (IC).....	5
10. PCR Protocol.....	6
11. Programming Rotor-Gene	6
12. Programming StepOne	7
13. Programming Other Machines	7
14. Data Analysis: Rotor-Gene.....	7
15. Data Analysis: StepOne	10
16. Quantitation	12
17. Linear Range	12
18. Sensitivity	12

HBV RQ kit is intended for use with Rotor-Gene or StepOne machines and for the quantitative detection of HBV-DNA extracted from plasma. This kit is for research use only.

1. Introduction

Hepatitis B, a viral infection with Hepatitis B virus (HBV) is a major global health issue. About 2 billion people are infected world wide of which 350 million are chronic carriers. In recent years, accessing the level of circulating virus in blood has been regarded as the most reliable tool for disease monitoring and patient management including clinical staging and pretreatment evaluation, monitoring and evaluation of antiviral therapy success and detection of emerging new resistant viruses.

HBV RQ kit provides a ready-to-use Real-Time PCR system for detection and quantitation of HBV DNA with Rotor-Gene or StepOne machines. Currently the Real-Time PCR provides the highest sensitivity and widest dynamic range among other methods. In this method application of fluorescent probes allows detection of amplified product. Analysis of fluorescent kinetics also leads to quantification of the target sequence in the reaction without requiring post-amplification analysis therefore, reducing the possibility of contamination with the PCR product. This kit also incorporates an *Internal Control* (IC) to identify possible DNA extraction failure or PCR inhibition.

2. Kit Contents

The kit contains a manual, a CD with Rotor-Gene and StepOne templates and following reagents:

Label	Content	Quantity
HBV Mix	PCR Master mix*	360 μ l
HBV S1	Standard 1: 100,000 IU/ μ l	150 μ l
HBV S2	Standard 2: 10,000 IU/ μ l	150 μ l
HBV S3	Standard 3: 1,000 IU/ μ l	150 μ l
HBV S4	Standard 4: 100 IU/ μ l	150 μ l
HBV S5	Standard 5: 10 IU/ μ l	150 μ l
HBV IC	Internal Control*	250 μ l
Water	PCR Grade Water	200 μ l

* 1, 2 and 4 tubes for 24, 48 and 96 reaction kits.

3. Storage and Stability

The kit components should be shipped and stored at -20°C and are stable until the expiry date mentioned. Avoid repeated freeze-thaws more than few times to prevent reduced sensitivity.

4. Additionally Required Materials

To use this kit, you need the following items:

- Real-Time PCR machine and accessory computer
- Table top microtube centrifuge
- Vortex Mixer
- Dry Block Heater
- Adjustable pipettors and nuclease free filtered tips
- DNA extraction kit
- Nuclease free 1.7ml microtubes and PCR microtubes
- Disposable powder-free gloves

- Cold block

5. General Precautions

In order to prevent false results, always pay attention to the following points:

- **Treat all samples as potentially infectious.**
- Within the pre-PCR work area assign three separate spaces for: a) Sample storage and extraction, b) Reagent preparation where the master-mix is aliquoted into tubes and c) Reaction preparation area for addition of extracted DNA to the tubes.
- Always wipe the working surfaces with 70% Ethanol before and after work.
- Thaw kit components on ice completely, mix by flickering followed by a quick spin and store on crushed ice after.
- Do not place 0.2ml PCR tubes on crushed ice. Use cold blocks instead.

6. Specimen, Storage and Transport

Peripheral blood should be collected in sterile condition in proper and sterile tubes. We recommend EDTA or citrate plasma for HBV detection. Whole blood or plasma should be shipped at +4°C. Upon receipt plasma should be separated from whole blood and can be stored at +4°C for few days or aliquoted and stored at -20°C for up to few weeks.

7. Interfering Substances

Heparin (more than 10 IU/ml) affects the PCR. Blood collected in heparin containing tubes should not be used. Samples of heparinized patients must not be used as well. Elevated levels of

bilirubin (≤ 4.5 mg/dl) and lipids (≤ 1000 mg/dl) and hemolytic samples do not influence the extraction and PCR.

8. DNA Isolation

DNA isolation can be performed using different kits from various manufacturers. We recommend the following:

- High Pure Viral Nucleic Acid Kit (Cat. no. 11858874001, Roche Applied Science, Mannheim, Germany)
- QIAamp DNA Blood Mini Kit (Cat. no. 51104, Qiagen GmbH, Hilden, Germany)
- QIAampUltraSens ® Virus Kit (Cat. no. 53704, Qiagen GmbH, Hilden, Germany)
- QIAampMiniElute Virus Spin Kit (Cat. no. 57704, Qiagen GmbH, Hilden, Germany).

To monitor DNA extraction process, internal control should be applied to the extraction process. For more details, please, refer to section 9 of this handbook.

9. Internal Control (IC)

In order to evaluate the possibility of DNA extraction failure and PCR inhibition and prevent false negative results, HBV RQ kit contains internal control (IC). IC can be used during extraction process or simply added to HBV Mix.

To monitor both DNA extraction and PCR reaction, internal control should be added to the mixture of lysis buffer and patient sample. Required volume of internal control is 10% of elution buffer. For instance if extracted DNA is eluted with 100ul, then 10ul of internal control should be added to the mixture of lysis buffer and patient sample. **Please note that internal control should not be added directly to the patient sample (i.e. before addition of lysis buffer) as it loses its efficiency.**

If IC is added to HBV Mix, only PCR inhibition can be monitored. For this purpose, 1ul of IC reagent should be added to each reaction. For example, for 10 PCR reactions, 15ul of internal control should be added to 150ul of HBV Mix before it is added to the tubes.

In a successful DNA extraction and PCR test, IC should generate a CT of 31-37 in Yellow/VIC Channel.

10. PCR Protocol

Thaw the reagents on ice completely followed by a brief mixing and a quick spin. Place required number of tubes on cold block. Consider one tube for each sample plus one for each standard and one for the negative control.

If IC is introduced during extraction process, pipette 15ul of HBV Mix to each PCR tube.

If IC is added to the HBV Mix, add 15ul of the prepared mix (as described in section 9) to each PCR tube.

Then add 10ul of extracted DNA, standard or water to each tube.

Cap the tubes and visually inspect to make sure all are capped securely. Place tubes in the machine.

Note: Working with StepOne instrument, spin tubes briefly before loading on the block.

Note: If using Rotor-Gene attach the locking ring.

11. Programming Rotor-Gene

Before you start the machine, make sure you have attached the locking ring on the rotor!

Open the CD provided in the kit and double click on HBV 0.2 or HBV-strip template depending on used tubes. Program starts.

Click on Start button (Green button on the top menu). On the pop up window click start again and save program on desired location.

12. Programming StepOne

Open the StepOne software (V 2.*). On the Set Up menu click on Template and select the file on CD provided with the kit. Click on Plate Setup. One negative control, 5 standards and 10 samples are defined. You may change plate set up using right click options (copy, paste, clear). You may also add /remove samples or change sample name on “Define Targets and Samples” menu. When finished, click on “Start Run” and save the experiment on desired location. Instrument will start shortly.

13. Programming Other Machines

If you apply this kit to other Real-Time PCR machines, program it according to the following table:

Step	Temperature and time	Cycles
1	95°C x 10 min	1 cycle
2	95°C x 15 sec	45 cycles
	60°C x 60 sec	

Fluorescence should be collected at 60°C for FAM and VIC/HEX dyes.

HBV Mix contains Rox with the final concentration of 300nM in the reaction.

14. Data Analysis: Rotor-Gene

Before analyzing results, make sure in the sample menu all the standards have been defined as “standard” and the relative concentrations have been entered. Patient samples should be

defined as "unknown" and Negative control or no template control as "Negative Control" or "NTC" respectively.

Analyze data according to manufacturer recommendations. Perform quantitative analysis for HBV (Green channel) and qualitative analysis for Internal Control (Yellow channel). Briefly, click on analysis menu and then under Quantitation tab double click on cycling A. Green.

In the pop up for Automatic Threshold increase the minimum or lower bound until it surpasses the negative control or the NTC fluorescence, and then click on OK. You may also, simply set the threshold at 0.1.

Repeat the above for Cycling A. Yellow but cancel the Automatic Threshold and manually set threshold on 0.1.

Figures 1 and 2 represent typical graphs for Rotor-Gene machine.

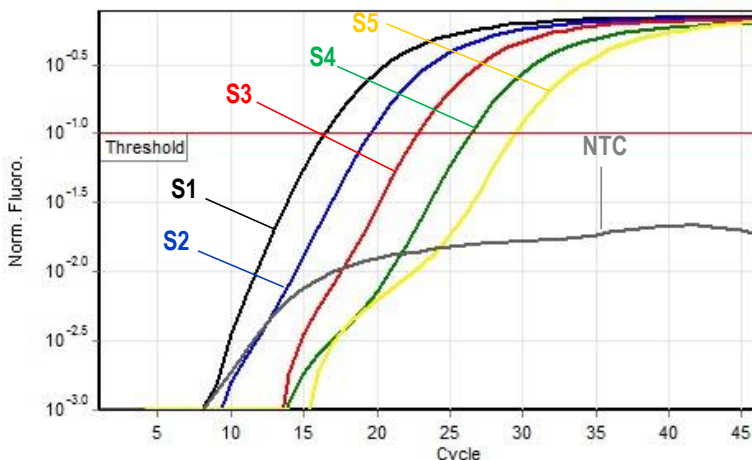
Figure 1: Typical HBV Graph in Green Channel for Rotor-Gene

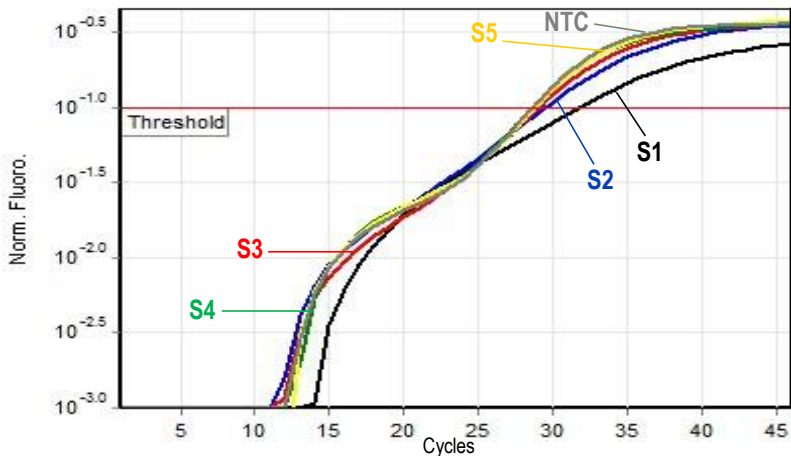
Figure 2: Typical IC Graph in Yellow Channel for Rotor-Gene

Note that a sample is considered Positive only if it has a log phase, and only then CT is reliable and can be used. In the absence of log phase, sample is considered Negative and CT if present is not reliable.

Consider following points when analyzing:

- A sample is **Positive** if it is positive in Green channel with a CT of less than 40. The viral load or quantitation results in the Cycling A. Green are valid.
- A sample is **Negative** if it is negative in Green channel while it is positive in Yellow channel with a sigmoid graph and CT of 28-34.
- Results are **Inconclusive** and the test should be repeated if a sample is negative in both of Green and Yellow channels.

15. Data Analysis: StepOne

Analyze data according to manufacturer recommendations. Briefly, click on Analyze and set the threshold for HBV/FAM at 0.1 and at 0.1 for IC/VIC.

Figures 3 and 4 represent typical graphs for StepOne machine.

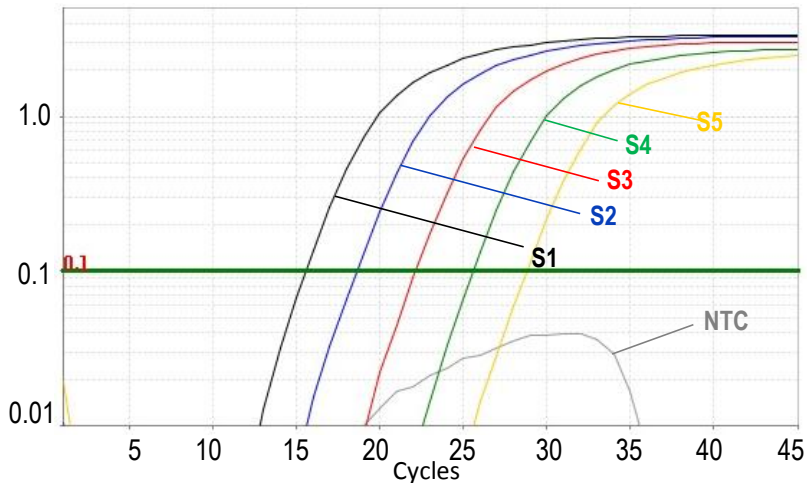
Figure 3: Typical HBV Graph in FAM Channel for StepOne

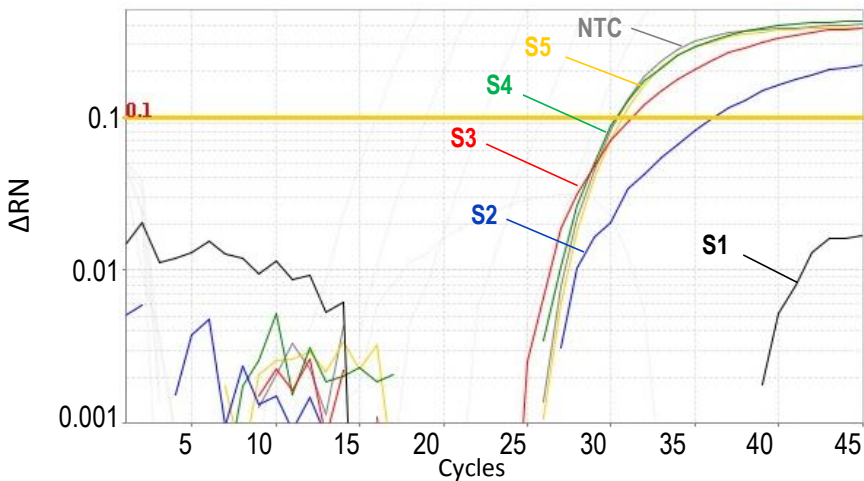
Figure 4: Typical IC Graph in VIC Channel for StepOne

Note that a sample is considered Positive only if it has a sigmoid graph and log phase, and only then CT is reliable and can be used.

In the absence of sigmoid graph and log phase, sample is considered Negative and CT if present is not reliable.

Consider following points when analyzing:

- A sample is **Positive** if it is positive in FAM/HBV channel with a CT of less than 40. The viral load or quantitation results are valid.
- A sample is **Negative** if it is negative in FAM/HBV channel while it is positive in VIC/IC channel with a CT of 28-34.
- Results are **Inconclusive** and the test should be repeated if a sample is negative in both of FAM/HBV and VIC/IC channels.

16. Quantitation

The kit provides 5 quantitation standards with defined titers to generate a standard curve for quantification of samples viral load. Working with Rotor-Gene machine, the standard curve from a previous run can also be imported for quantification of samples to the recent run. To do so, at least one standard must be used in the current run. Apparently using all five standards in each run will lead to results that are more accurate.

Quantitation standards are defined as IU/ul. To convert the result to IU/ml following equation should be used:

$$\text{Result(IU/ml)} = \frac{\text{Result(IU/}\mu\text{l)} \times \text{elution volume}(\mu\text{l})}{\text{sample volume(ml)}}$$

“Sample volume” is the plasma volume used for DNA isolation and “Elution volume” is the volume of buffer or water used to elute or dissolve isolated DNA.

17. Linear Range

The linear range of the kit was assessed with dilution series of the cloned target and the assay showed to be linear in the range of 10,000,000 IU/μl to 1 IU/μl.

18. Sensitivity

The analytical detection limit of the kit was assessed with dilution series of the cloned target and the detection limit was determined as 0.15 IU/μl.